


TRUST® INSTRUCTION FOR USE

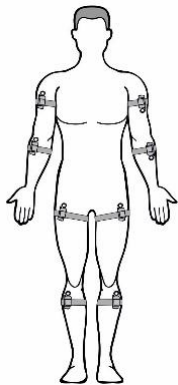


Place the TRUST tourniquet over the injured limb 5-10 cm above the wound or in the highest possible position of the limb.

Attention! Never place the tourniquet on the joints or wound!


1

Place the TRUST 5-10 cm above the wound

Two-handed application

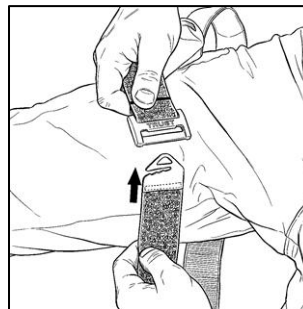

One-handed application


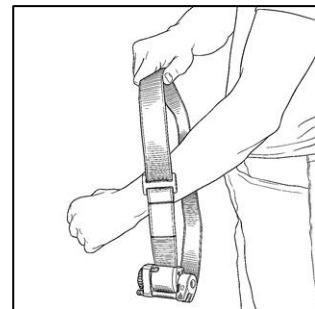


Pull the band end in the opposite direction from the injured limb until the band loop is fully tightened and fasten the band in the opposite direction on itself.

- Ensure the tightening rod is opened and the tourniquet unit is within reach of your healthy hand;
- Fasten the band over the housing of tourniquet for double safety.

2

Pull and fasten the band around a limb and housing

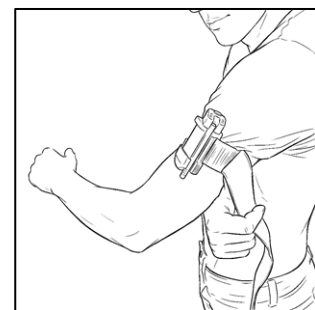
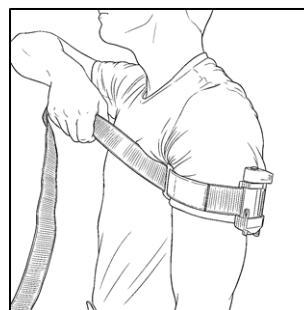


Tighten the band by rotating the handle back and forth in reciprocating rotary motions several times **until the bleeding stops.**

- To increase torque, press your thumb against the housing rest;
- The TRUST tourniquet equipped with a ratchet mechanism with an automatic lock.

3

Tighten the band by rotating handle back and forth

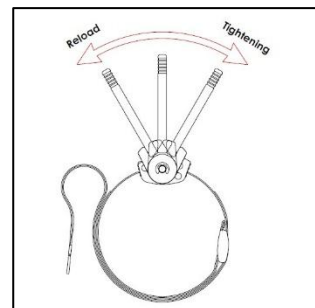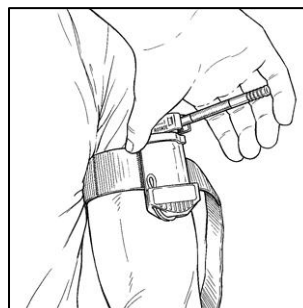



! Control the bleeding !

4

Secure the handle and record time

Secure the handle to the socket on the housing;

Record the time of applying of the tourniquet on the Time plate.

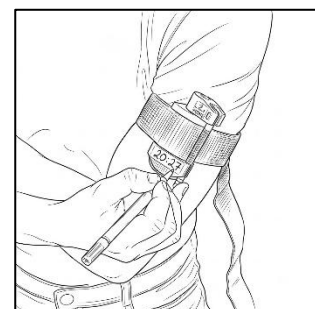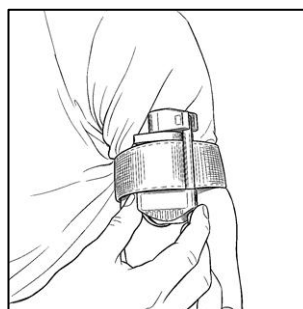

GRADUAL RELEASE OF THE TOURNIQUET


To gradually release the band, the tourniquet is equipped with a ratchet release system using a loop. **Firstly, open the handle and unfasten the band just till the housing, then**

- 1) Press the handle,
- 2) Pull release loop and
- 3) Slowly release the handle.

Control the bleeding!

If bleeding resumes, tighten the tourniquet again.

1) Press

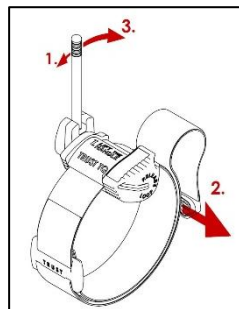

2) Pull



3) Slowly release


UNWINDING THE BAND (RESET)

In case, when necessary, fully **unwind the band**, fold the handle away from the housing, pull the release loop outward, and move the housing away from the limb, so that the hub with the handle rotates counterclockwise releasing the band's tension.


REMOVING THE TOURNIQUET

To completely remove the tourniquet from the limb, unfasten the band up to the sliding buckle and pull the end of the band out of the buckle opening.


Release and unwind

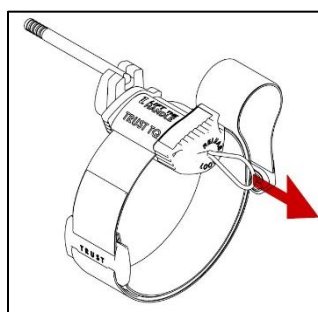

Unfasten and remove

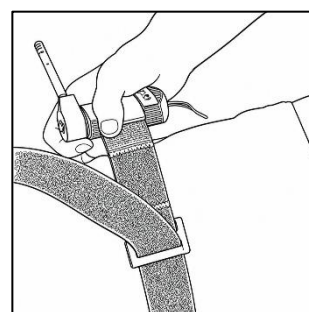

SPECIFICATION AND DESCRIPTION

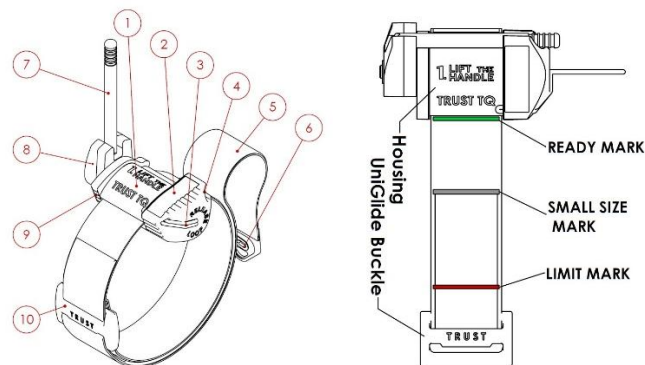

1. Housing, 2. Time plate, 3. Release loop,
4. Handle socket, 5. Band, 6. Band hook,
7. Tightening handle, 8. Handle hub,
9. Thumb rest, 10. UniGlide Buckle

TACTICAL
BLACK



TRAINING
BLUE



HIGHLY
VISIBLE
ORANGE



CUSTOM



MANUFACTURER INFORMATION



SIA "AGM Orthopaedics"
Registration number: 42103076971
46 Graudu Street, Liepaja LV-3401, Latvia
SRN-Eudamed: LV-MF-000044288

For more information
visit us at
www.TRUST-TQ.com



TRUST® is a registered trademark in the European countries.
©2026 AGM Orthopaedics. All rights reserved.
Revision 3.27.00 | 16.07.2026